

Medical device

30 soft sterile disposable eye wipes

Soft sterile disposable eye wipe, soaked in an isotonic, water-based solution of active ingredients selected for their proven efficacy. Suitable for daily use.

INDICATIONS

- Cleansing, moisturization and decongestion of the inner and outer eyelids and the periocular area.
- Ocular hypersensitivity caused by environmental factors such as dust, smog, pollen or prolonged exposure to screens. Counters redness, tearing and itching.
- Reduction of dryness and foreign body in the eyes feeling.
- Adjuvant in case of allergic or infectious eye diseases (conjunctivitis, blepharitis, stye, chalazion), problems associated with thyroid disorders, infections of various types, ophthalmic symptoms of conditions such as diabetes and deficiency of vitamins A, B and PP in combination with therapies indicated by the specialist.
- Cleansing and moisturising before and after ophthalmic surgeries.
- Cleansing and redness reducer for contact lens wearers.

COMPOSITION

Sodium hyaluronate, Chamomilla recutita extract, Helichrysum italicum extract, Polysorbate 20, Lauryl glucoside, Linoleamidopropyl pg-Dimonium Chloride Phosphate.

- **Hyaluronic acid (Sodium hyaluronate)**: has moisturising, humectant and soothing properties.
- **Linoleamidopropyl pg-Dimonium Chloride Phosphate (phospholipid)**: has a cleansing action.
- **Chamomilla recutita**: has soothing, decongestant and refreshing, as well as anti-inflammatory, anti-redness and anti-itching properties.
- **Helichrysum italicum**: has anti-inflammatory/antioxidant (photo-protective action against UV radiation) and decongestant properties.

INSTRUCTIONS FOR USE

Self-use: open the sachet using the easy opening.

Extract the wipe and gently wipe inside the eyelid and around the periocular area.

Use a new wipe for each eye, and replace it after cleansing an infected area, to avoid contamination of other areas.

Do not rinse or dry the affected area.



Professional use: the easy-opening sachet is designed for use in sterile environments.

Extract the wipe and gently wipe inside the eyelid and around the periocular area.

Use a new wipe for each eye, and replace it after cleansing an infected area, to avoid contamination of other areas.

Do not rinse or dry the affected area.

WARNINGS

In children, it is advisable to use the wipes externally only, to clean the eyelashes and periocular area when the eyes are closed.

Tissue tolerance and biocompatibility:

- LinfoVir® Ofta does not burn or cause pain;
- LinfoVir® Ofta does not irritate and is not a sensitiser;
- LinfoVir® Ofta is compatible with other ophthalmic medications.

GENERAL SAFETY INSTRUCTIONS AND WARNINGS

The sterile product is suitable for cleaning the inside of the eyelid and the periocular area. Take particular care when using the product inside the eyelid. Use intact and undamaged packs only. Store in a cool and dry place. Keep away from direct heat sources and/or direct sunlight. Use only within the period of validity, pay attention to the expiry date.

Rev: January 2021



» Scan the QR Code with your smartphone



» Explore the entire Nóos product line



Gastrointestinal



Respiratory



Immunity



Hearing



Oral cavity



Eye care



Rare diseases

Welcare Industries S.p.A.
Via dei Falegnami, 7 - 05018 Orvieto (TR), Italy
www.welcareindustries.com

Distributed by:

 **NÓOS** Via Campello sul Clitunno, 34 - 00181 Roma
www.noosif.com

NÓOS FOR THE ENVIRONMENT
PAP 22 LEAFLET PAPER

Check your local municipal guidelines