

InfoVir[®] Iperwash

Medical device

8 single-dose 60 ml bottles
+1 nasal dispenser

InfoVir[®] Iperwash is a medical device based on a pH6 buffered hypertonic saline solution, consisting of a mechanical system specially designed to clear and sanitise the nasopharyngeal cavity through the nostrils, performing an osmotic and decongestant action.

INDICATIONS

InfoVir[®] Iperwash is a hypertonic saline solution useful in all those cases of congestion nasopharyngeal and inflammation of the nasal cavities with accumulation and stagnation of mucus. It generates a sanitising jet able to clean thoroughly and to help restoring the patency of the upper airways and to favour breathing.


COMPOSITION

Purified water, sodium chloride, dibasic potassium phosphate, monopotassium phosphate.
Preservative free.

HOW IT WORKS

Its special nosepiece allows the nostril to be completely closed, producing, at bellow pressure, a sanitising jet, which reaches the nasopharynx and, by activating the palatal reflex, comes out of the opposite nostril.

HOW TO USE

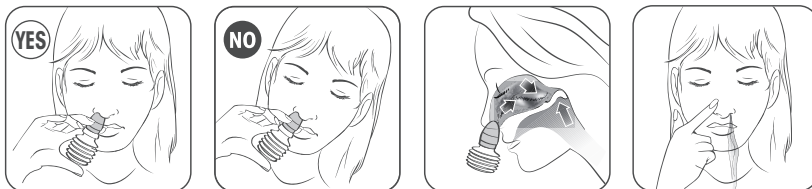

1. Before using the product, blow your nose well.
2. Open a bottle and pour about half of its content into the dispensing bellow. Then close the bellow with the blue nosepiece, pushing it all the way until it comes into contact with the base of the ferrule.
3. Rest the dispenser on the nostril, taking care to create a perfect adherence that allows it to be completely closed.
4. Press the bellow all the way down to administer the product.
5. Use the second half of the saline solution to flush through the other nostril.

HOW AND WHEN TO USE IT

Use the product twice a day (morning and evening), as needed or on medical advice. Each single-dose bottle contains enough solution for a complete wash out of both nostrils. If deemed appropriate, hold the bellow in your hand slightly warming the solution before administering it.

STORAGE CONDITIONS

Store in a cool and dry place away from heat sources. Do not use after the expiry date indicated on the package. The expiry date refers to the product intact and correctly stored.

WARNINGS AND PRECAUTIONS

For the use in children under 2 years of age, seek medical advice. Do not use in patients with ear trauma. Do not use in case of proven sensitivity to the components of the solution. The use of the dispenser is strictly personal. Use the solution only for the treatment of the upper airways and the dispenser only with the supplied solution. Not for injection.

PACKAGE CONTENTS

- 1 bellow dispenser
- 1 soft blue nosepiece, adaptable to all nostrils
- 8 single-dose bottles of LinfoVir® iperwash hypertonic saline solution buffered at pH6

Latest revision of the instructions for use: May 2021

Tell us about your experience

» Scan the QR Code to answer the questionnaire

» and to discover the entire NÓOS product line



Gastrointestinal



Respiratory



Immune



Hearing



Oral cavity



Eye care



Rare diseases



Via Campello sul Clitunno, 34 - 00181 Roma
www.noosif.com

Manufacturing site: Biofarma S.r.l.
Via Castelliere, 2 - 33036 Mereto di Tomba (UD)



NÓOS FOR THE ENVIRONMENT

PAP 22

LEAFLET

PAPER

Check your local municipal guidelines