

Oracol[®] gel

Medical device

6 x 3 ml ampoules

INDICATIONS

Oracol[®] gel is a medical device useful in case of irritation and inflammation of the oral cavity. Helps in providing relief from pain caused by oral mucosal lesions of various origins: stomatitis, gingivitis, mouth ulcers, trauma. Forms a protective film that, adhering to the mucosa, insulates the wound from the external environment and keeps it in pH and moisture levels ideal for faster repair.

INSTRUCTIONS FOR USE

Apply a thin layer of gel to the injured oral mucosa, where necessary spreading the gel with a swab or clean and dry fingers to ensure a correct and even distribution. Repeat the treatment at least twice a day after meals, having previously carried out appropriate oral hygiene. Continue applying until the symptoms disappear.

INGREDIENTS

Aqua, Colostrum Whey, Xylitol, PEG-40 Hydrogenated Castor Oil, Cellulose Gum, PVP, Phenoxyethanol, Chondrus Crispus, Diazolidinyl Urea, Aroma, Sodium Dehydroacetate, Lactic Acid, Folic Acid.
Gluten-free

WARNINGS/PRECAUTIONS

After each use, it is recommended not to eat or drink, to prevent an early removal of the product. Oral use. Harmless if accidentally ingested. It is recommended not to reuse the ampoule once it has been opened. **Keep out of the reach of children.**

CONTRAINDICATIONS/SIDE EFFECTS

Biocompatibility studies have not revealed any contraindications or undesirable effects; however, use of the product should be avoided in patients with a history of hypersensitivity to any of its components. If in doubt, consult your doctor before using the product.

STORAGE CONDITIONS

Store at room temperature, away from heat sources. The expiry date indicated on the ampoules refers to the unopened product, correctly stored. Do not use the product after this date.

Instructions for use last reviewed: 13 July 2018



 **Manufacturer:** Difa Cooper Spa
Via Milano 160 - 21042 Caronno Pertusella (VA) Italy

Distributed by:



Via Campello sul Clitunno, 34 - 00181 Roma
www.nooos.it

NÓOS FOR THE ENVIRONMENT

PAP 22

LEAFLET

PAPER

Check your local municipal guidelines

»» Scan the QR Code with your smartphone



»» Explore the entire Nóos product line



Gastrointestinal



Respiratory



Immunity



Hearing



Oral cavity



Eye care



Rare diseases