



Medical device

8 single-dose 60 ml bottles
+1 nasal dispenser

LinfoVir[®] isowash is a medical device based on isotonic saline solution, consisting of a mechanical system specially designed to clear and sanitise the nasopharyngeal cavity through the nostrils.

INDICATIONS

LinfoVir[®] isowash is an isotonic saline solution for the daily hygiene of the nasopharynx. It generates a sanitising jet capable of thoroughly cleaning while respecting the physiology of the nasal mucosa and maintaining the correct hydration.


COMPOSITION

Purified water, sodium chloride 0.69%, dibasic potassium phosphate 0.36% monopotassium phosphate 0.15%. **Preservative free.**

HOW IT WORKS

Its special nosepiece allows the nostril to be completely closed, producing, at bellow pressure, a sanitising jet, which reaches the nasopharynx and, by activating the palatal reflex, comes out of the opposite nostril.

HOW TO USE

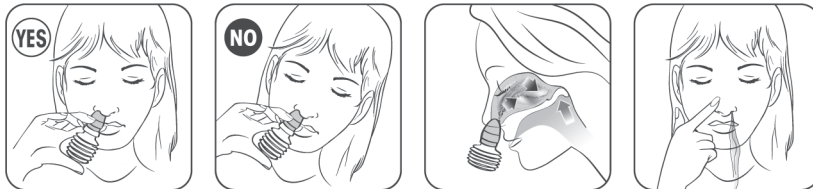

1. Before using the product, blow your nose well.
2. Open a bottle and pour about half of its content into the dispensing bellow. Then close the bellow with the blue nosepiece, pushing it all the way until it comes into contact with the base of the ferrule.
3. Rest the dispenser on the nostril, taking care to create a perfect adherence that allows it to be completely closed.
4. Press the bellow all the way down to administer the product.
5. Use the second half of the saline solution to flush through the other nostril.
6. At the end of the procedure, remove the nosepiece from the dispensing bellow, wash both components carefully with water, taking care to eliminate any residues of saline solution which may, in the long run, alter the characteristics of the dispensing device. Leave to air dry and store everything in a clean and dry place.

HOW AND WHEN TO USE IT

Use the product once a day or on medical advice. Each single-dose bottle contains enough solution for a complete wash out of both nostrils. If deemed appropriate, hold the bellow in your hand slightly warming the solution before administering it.

STORAGE CONDITIONS

Store in a cool and dry place away from heat sources. Do not use after the expiry date indicated on the package. The expiry date refers to the product intact and correctly stored.

WARNINGS AND PRECAUTIONS

For the use in children under 2 years of age, seek medical advice. Do not use in patients with ear trauma. Do not use in case of proven sensitivity to the components of the solution. In case of adverse reaction, stop using the product and seek medical advice. The use of the dispenser is strictly personal. Use the solution only for the treatment of the upper airways; use the dispenser only with the supplied solution. Single use medical device.

Not for injection. Keep out of reach of children. For pregnant or breastfeeding women it is recommended to seek medical advice. Do not use if the packaging is opened or damaged. It is recommended not to use drugs or other medical devices before or immediately after the wash, to avoid that they may be wash out together with the secretions. It is necessary to report any serious incident that has occurred in relation to the device to the manufacturer and to the Competent Authority of the Member State in which the user and/or patient is established.

SIDE EFFECTS

There are no known side effects. To date, there are no known interactions with other devices, substances or medicines.

PACKAGE CONTENTS

- 1 bellow dispenser
- 1 soft blue nosepiece, adaptable to all nostrils
- 8 single-dose bottles of LinfoVir® isowash isotonic saline solution

Latest revision of the instructions for use: Rev 02 of 07/2023

Tell us about your experience

» Scan the QR Code to answer the questionnaire



» and to discover the entire Nóos product line



Gastrointestinal



Respiratory



Immune



Hearing



Oral cavity



Rare diseases



Via Campello sul Clitunno, 34 - 00181 Roma
www.noosit.com



Consult the guidelines for your city