

# InfoVir<sup>®</sup> plus

nasal drops

Medical device

20 ml bottle with measuring cap

LinfoVir<sup>®</sup> Plus nasal drops is a medical device containing Resveratrol from *Polygonum cuspidatum* and carboxymethyl beta glucan in buffered saline solution. The product, through topical nasal use, may be useful in subjects highly prone to inflammations and/or infections of the nasal cavities.

## INDICATIONS

Medical device useful for favouring the fluidification of nasal secretions and, consequently, their removal, thus favouring breathing. Adjuvant in the treatment of inflammatory forms of the nasal cavities. Suitable also in children under 3 years of age.

## COMPOSITION

Purified water, carboxymethyl beta glucan 0.0065%, *Polygonum cuspidatum* rhizome titrated at 98% in resveratrol 0.0504%, dibasic potassium phosphate, mannitol, propylene glycol, sodium chloride 0.2292%, potassium phosphate, nipagin sodium.

## HOW TO USE

2-3 drops per nostril 3 or 4 times a day.

When using the product for the first time, fully screw the bottle cap clockwise as far as it goes, so that the powder contained in the cap falls into the liquid. Shake vigorously and replace the cap with the special dropper contained in the package, screwing it down.



Instill in the nostrils. Shake before each use. After using the product, clean the dropper and close the bottle with the cap. The presence of Resveratrol (extracted from the *Polygonum cuspidatum* plant) can determine a slightly amber colour of the product without compromising its quality.

## WARNINGS

Do not use in case of known hypersensitivity to the product ingredients. In case of adverse reactions, discontinue the use of the product and consult your doctor. Close the device well after use. Do not use after the expiration date indicated. The reconstituted product is stable for 28 days. Use the reconstituted product within 28 days. Keep out of the reach of children. The use of the bottle is strictly personal. For pregnant or breastfeeding women it is recommended to seek medical advice. Do not use if the packaging is opened or damaged. It is recommended not to use drugs or other medical devices before or immediately after using the product, to avoid that they may be removed together with the secretions. It is necessary to report any serious incident that has occurred in relation to the device to the manufacturer and to the Competent Authority of the Member State in which the user and/or patient is established.

## SIDE EFFECTS

There are no known side effects. To date, there are no known interactions with other devices, substances or medicines.

## PACKAGING

20 ml bottle with measuring cap and dropper.

Latest revision of the instructions for use: Rev 1 of 11/2023

*Tell us about your experience*

» Scan the QR Code to answer the questionnaire



» and to discover the entire Nóos product line



Gastrointestinal



Respiratory



Immune



Hearing



Oral cavity



Rare diseases



Via Campello sul Clitunno, 34 - 00181 Roma  
[www.noos.it](http://www.noos.it)



NÓOS FOR THE ENVIRONMENT

PAP 22

LEAFLET

PAPER

Consult the guidelines for your city

