

# Oracol<sup>®</sup> gel

6x3 ml ampoules

## INDICATIONS

Adjuvant for the protection of oral mucosa and gums in case of irritation. Helps in providing relief from pain caused by stomatitis, gingivitis, mouth ulcers, trauma. Thanks to the thickening and filming properties of PVP, it creates a protective layer which, adhering to the mucosa, isolates it from the external environment. The presence of folic acid keeps the mucosa in pH and humidity conditions ideal to favour the normal healing process.

## INSTRUCTIONS FOR USE

Apply a thin layer of gel spreading it with a swab or clean and dry fingers. Repeat the application at least twice a day, better in the morning and in the evening, after meals, having previously carried out appropriate oral hygiene. After each application it is recommended not to eat or drink, to prevent an early removal of the product.

## WARNINGS

Keep out of the reach of children. Avoid contact with eyes. Do not ingest. In case of hypersensitivity to any of the components consult your doctor before using the product. The use of the ampoule is strictly personal.

## STORAGE

Store at room temperature away from heat sources and humidity. Once opened, the ampoule can be stored in fridge for maximum 3 months. The expiry date refers to the unopened product correctly stored.

» Scan the QR Code with your smartphone



» Explore the entire Nóos product line



Gastrointestinal



Respiratory



Immunity



Hearing



Oral cavity



Rare diseases

**Difa Cooper S.p.a.**

Via Milano 160 - 21042 Caronno Pertusella (VA) Italy

Distributed by:



Via Campello sul Clitunno, 34 - 00181 Roma  
[www.noosif.com](http://www.noosif.com)



Check your local municipality guidelines